## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USE/INDICATIONS:** Immobilization and support of the arm, shoulder and elbow joints.

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the support over an open wound.

### APPLICATION INFORMATION:

NOTE: Universal Arm Sling envelope length may be shortened if necessary by folding the open end inside itself and securing the hook and loop closures. Apply as noted below.

- 1) Position elbow in the pocket of the sling. The wrist and 3/4 of the hand should be supported within the pocket.
- 2) If your sling pocket has a thumb loop, you may place thumb in loop to help maintain position of forearm.
- 3) Place shoulder straps over opposite shoulder, feed through the D-ring and secure. Hand should be slightly elevated above elbow.

### CLEANING INSTRUCTIONS:

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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